

How to Use CLEAR CARE® and CLEAR CARE® PLUS Solutions

They work a little differently than what you might be used to. But that's what makes us awesome! Just follow these simple steps and you'll be good to go.

1

WASH, RINSE
—AND—
DRY YOUR
HANDS



* NOTE: These instructions are for soft contact lenses only. If you are using CLEAR CARE® Solution and have RGP lenses, [click here](#) for specific instructions.

2

PLACE LENSES IN
BASKETS
—AND—
RINSE FOR
5 SECONDS
WITH SOLUTION



3

FILL THE
LENS CASE
—TO—
THE LINE
WITH THE
SOLUTION

! RED CAP ALERT!
That red cap means you should never put CLEAR CARE® or CLEAR CARE® PLUS Solution directly in or around your eyes.



4

PUT IT
ALL TOGETHER
—AND—
TIGHTEN
THE LID



5

—SOAK—
OVERNIGHT
OR FOR AT LEAST
6 HOURS

! WARNING!
You MUST wait a full 6 hours! If you don't, you may hurt your eyes.